

Urgent Field Safety Notice

AIMC 25-06.B.OUS

Atellica CI Analyzer
Atellica IM Analyzer
ADVIA Centaur XP System
ADVIA Centaur XPT System
ADVIA Centaur CP System

Title	Atellica IM and ADVIA Centaur Homocysteine Calibrator Instability
Date Issued	Oct-2025

Products	Product	Siemens Material Number/Unique Device Identification	Kit Lot Number	Manufacturing Date	Expiration Date
	Atellica IM Homocysteine Calibrator (2 Pack)	10995498 / 00630414598253	25675A77	1-Apr-2025	1-Apr-2026
	ADVIA Centaur HCY Calibrator (2 Pack)	10310376 / 00630414468198	25674A77		

Issue Description Siemens Healthineers has confirmed through internal investigation that the Atellica IM Homocysteine Calibrator and ADVIA Centaur HCY Calibrator Lots listed in the table above do not meet the calibrator stability claim throughout the 12-month shelf life. As the calibrator ages, homocysteine results were observed to increase with bias up to 13.1% at approximately 5 months post manufacture as compared to Siemens Internal Standardization.

The described calibrator instability impacts homocysteine on the Atellica CI Analyzer, Atellica IM Analyzer, ADVIA Centaur XP System, ADVIA Centaur XPT System, and ADVIA Centaur CP System.

Impact to Results Erroneously elevated homocysteine may occur as a result of this issue. A positive drift may impact both QC and patient results in all sample matrices.

Based on investigation results, users may observe elevated homocysteine results as compared to historical patient results and/or the published or laboratory-defined reference interval. Results of this assay should always be interpreted in conjunction with the patient's medical history, clinical presentation, and other findings.

Customer Actions

- Perform the instructions provided below:
 - Please review this letter with your Medical Director to determine the appropriate course of action, including for any previously generated results, if applicable.
 - Discontinue use of and discard the kit lots listed in the table above (Products Section).

-
- Review your inventory of these products to determine your laboratory's replacement needs and to provide information to Siemens Healthineers for reporting to the authorities.
 - Complete and return the Field Correction Effectiveness Check Form attached to this letter within 10 days.
 - Please retain this letter with your laboratory records and forward this letter to those who may have received this product.
-

Resolution Atellica IM and ADVIA Centaur Homocysteine Calibrator Lot ending in 77 is not available for purchase. Alternate Atellica IM and ADVIA Centaur Homocysteine Calibrator lots are available at Siemens Distribution Center(s).

**Single Registration
Number (SRN)** US-MF-000016560

We apologize for the inconvenience this situation may cause. If you have any questions, please contact your Siemens Healthineers Customer Care Center or your local Siemens Healthineers technical support representative.

Atellica and ADVIA Centaur are a registered trademark of Siemens Healthcare Diagnostics Inc. © Siemens Healthcare Diagnostics Inc. 2025.

FIELD CORRECTION EFFECTIVENESS CHECK

This response form is to confirm receipt of the enclosed Siemens Healthineers Urgent Field Safety Notice AIMC 25-06.B.OUS dated Oct-2025. Please read each question and indicate the appropriate answer.

If you have received any complaints of illness or adverse events associated with the products listed in the table on Page 1 immediately contact your local Siemens Healthineers Customer Care Center or your local Siemens Healthineers technical support representative.

Return this completed form as per the instructions provided at the bottom of this page.

- 1. Have you read and understood the instructions provided in this letter? Yes No
- 2. Do you have the affected product(s) on hand? Please check inventories before answering. Yes No
- 3. Were affected Site Personnel notified? Yes No
- 4. Was a copy of the letter retained and posted with the current product labeling? Yes No

If the answer to the question #2 above is yes, please complete the table below to indicate the quantity of affected product in your laboratory and replacement product required.

Product Description SMN #/Lot #	Quantity of Affected Product in inventory Discarded/Replacement Quantity Required		
<i>Atellica IM Homocysteine Calibrator (2PK) SMN: 10995498 Kit Lot: 25675A77</i>			
<i>ADVIA Centaur HCY Calibrator (2PK) SMN: 10310376 Kit Lot: 25674A77</i>			
Name of person completing questionnaire:			
Title:			
Institution:			
Street:			
City:	State:	Zip Code:	
Phone:	Country:		

Please send a scanned copy of the completed form via email to: fscreportingunit.my@siemens-healthineers.com

We apologize for the inconvenience this situation may cause. If you have any questions, please contact your Siemens Healthineers Customer Care Center or your local Siemens Healthineers technical support representative.